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Devises description

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The Teosyal®Pen is a new wireless and motorized injection system powered by a lithium battery.

The Teosyal®Pen is a non-sterile device intended to assist medical practitioners in injecting dermal fillers.

After inserting the Teosyal®Pen into the Syringe Holder (TeoLink), the practitioner simply presses the top of the TeoLink for a smooth and even delivery of the product, in continuous flow mode or in drops.

The box of Teosyal®Pen does not include a battery cap or TeoLinks.

The Teosyal Pen is only intended for Dermal Filler injections.

The TeoLink

The TeoLink is a syringe holder that has been designed to improve syringe handling ergonomics, allowing the user to have greater freedom to operate during injections.

The TeoLink's cover touch system provides touch activation along the entire upper surface of the syringe holder.

The TeoLink is compatible with most 1ml Hyaluronic Acid syringes on the market.

The TeoLink is compatible with either needles or cannulas.

The TeoLink is compatible exclusively with Teosyal®Pen.

Do not touch the key board while inserting the battery cap to the Teosyal®Pen.

The TeoLink box contains 25 TeoLink units + 1 battery cap.

Advantages of the Teosyal®Pen

For the patient :

- Douleur considerably less pain
- Highly precise injections
- Reduced side effects

For the practitioner :

- User-friendly & ergonomic design for more comfort
- Easy to use & intuitive system
- Optimized control of the procedure : depth of the injection, homogeneous delivery and precise dosage.

Precautions and recommendations

Please read the following instructions carefully and follow the proper safety measures in order to perform a successful injection.

- Only a qualified and experienced medical practitioner should use the Teosyal®Pen.
- The use of the Teosyal®Pen and the results thereof are the exclusive responsibility of the practitioner.

The practitioner is the only one who can evaluate the indications and contraindications relative to the technique to employ, the product to be administered and the appropriate needle to use.

In all cases, the medical practitioner shall proceed only after having fully familiarized himself or herself with both the product and its educational material.

Select the delivery mode and set-up the delivery speed or drop size only. Do not reuse or re-sterilize the equipment. Discard immediately after use in a separate waste container. Do not use if the package is opened or damaged.

Adverse reactions and contraindications

It is the practitioner's full responsibility to read and follow those instructions for use in order to avoid any adverse reactions or complaints.

Possible side effects and contraindications can result from the use of products in conjunction with the Teosyal®Pen (e.g. dermal filler, needle, cannula, etc.), it is the practitioner's full responsibility to check and follow the instructions for use of any products used with the Teosyal®Pen

General recommendations

- Keep the Teosyal®Pen out of reach of children.
- The equipment cannot be used where there is an inflammable mixture of air, oxygen or nitrogen.
- Store the Teosyal®Pen away from heat and sun exposure.
- Keep the Teosyal®Pen away from water and chemical products.
- Always check whether the device is functioning properly before use.
- If the Teosyal®Pen is damaged, it must be repaired or replaced by Teoxane SA. (please refer to Warranty).
- If the ON/OFF light turns orange, change the battery.
- The Teosyal®Pen must be returned to Teoxane SA when not in use, after 4 minutes.

The syringe holders (TeoLink), which are designed for use with the Teosyal®Pen exclusively and therefore cannot be used with any other TeoLink or button device.

Do not use other disposal cap than the ones supplied by Teoxane SA. Damage of the Teosyal®Pen caused by third party batteries voids the warranty.

Do not alter or modify any of the components: Teosyal®Pen, TeoLink (syringe holder) and battery.

Wearing single-use gloves is highly recommended for the use of the equipment.

No alteration of this equipment is allowed.

Dispose of the TeoLink (syringe holder) or battery according to your local environmental regulations.

Do not dispose of the Teosyal®Pen, but rather, return it to Teoxane SA, stating the reasons for the return.

Do not use the Teosyal®Pen near electronic equipment which emits electromagnetic fields.

Accessories

Battery cap

Button Battery Lithium 6B /170 mAh

The Teosyal®Pen must be used with a certified battery IEC60086-4 only

TeoLink

TeoLink (syringe holder) : Plastic PC Makrolon 2458

Cleaning

After each use, the Teosyal®Pen must be cleaned with an adequate cleaning agent before re-using it. Do not immerse the device or its components in water.

We recommend using an appropriate cleaning product in accordance with your local regulation.

Handling conditions

It is recommended to store and transport the Teosyal®Pen in its original packaging. The Teosyal®Pen box can also be used as a display.

Operating conditions 0 - 2000 m

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Humidity : 60% RH AT 30 % non-condensing

Transport conditions

Temperature : -20 °C TO 40 °C

Humidity : 60% RH AT 30 % non-condensing

Storage conditions

Temperature : 20 °C TO 40 °C

Humidity : 60% RH AT 30 % non-condensing

Assembly and use

Please refer to the assembly instructions drawings attached.

1. Unplug the black button of the Teosyal®Pen
2. Insert the disposable cap without touching the key board of the Teosyal®Pen.

3. Screw the battery cap
4. Unscrew the plunger rod from the syringe
5. Discard the plunger rod
6. Remove the finger grip from the syringe
7. Discard the finger grip
8. Insert the syringe into the TeoLink
9. Connect the TeoLink to the Teosyal®Pen
10. "Click" indicates the proper attachment of the TeoLink to the Teosyal®Pen

11. Remove and discard the cap from the syringe
12. Attach the needle to the syringe
13. Remove the needle protection. Do not discard it
14. Press the ON/OFF button. The green light indicates that the device is switched ON

15. Select the delivery mode and set-up the delivery speed or drop size
16. By pressing the top of the TeoLink mode the lead screw moves forward until it reaches the plunger
17. The injection can be performed by pressing the top of the TeoLink

After the use

18. Replace the needle protection
19. Pressing on both sides of the TeoLink release the syringe holder with the syringe and needle
20. Discard the TeoLink, the syringe and the needle
21. Press the reverse button to return the lead screw to its original position

The Device Control Buttons and Control Lights

1. Activation Button
2. Device ON/OFF button and light indicator
3. Continuous flow mode speed set up button and light indicator
4. Drop mode drop size set-up button and light indicator
5. Original position screw button and light indicator

Warranty

The warranty period for the Teosyal®Pen is 18 months from the date of purchase.

Note

Teoxane SA is not responsible for the cost of replacement of unused injectable material or injectable material lost or wasted during set-up, operation or maintenance of the system.

Please, contact Teoxane SA or your local representative for any further questions.

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